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
# Bar- & Ballabutment, indirect Workflow

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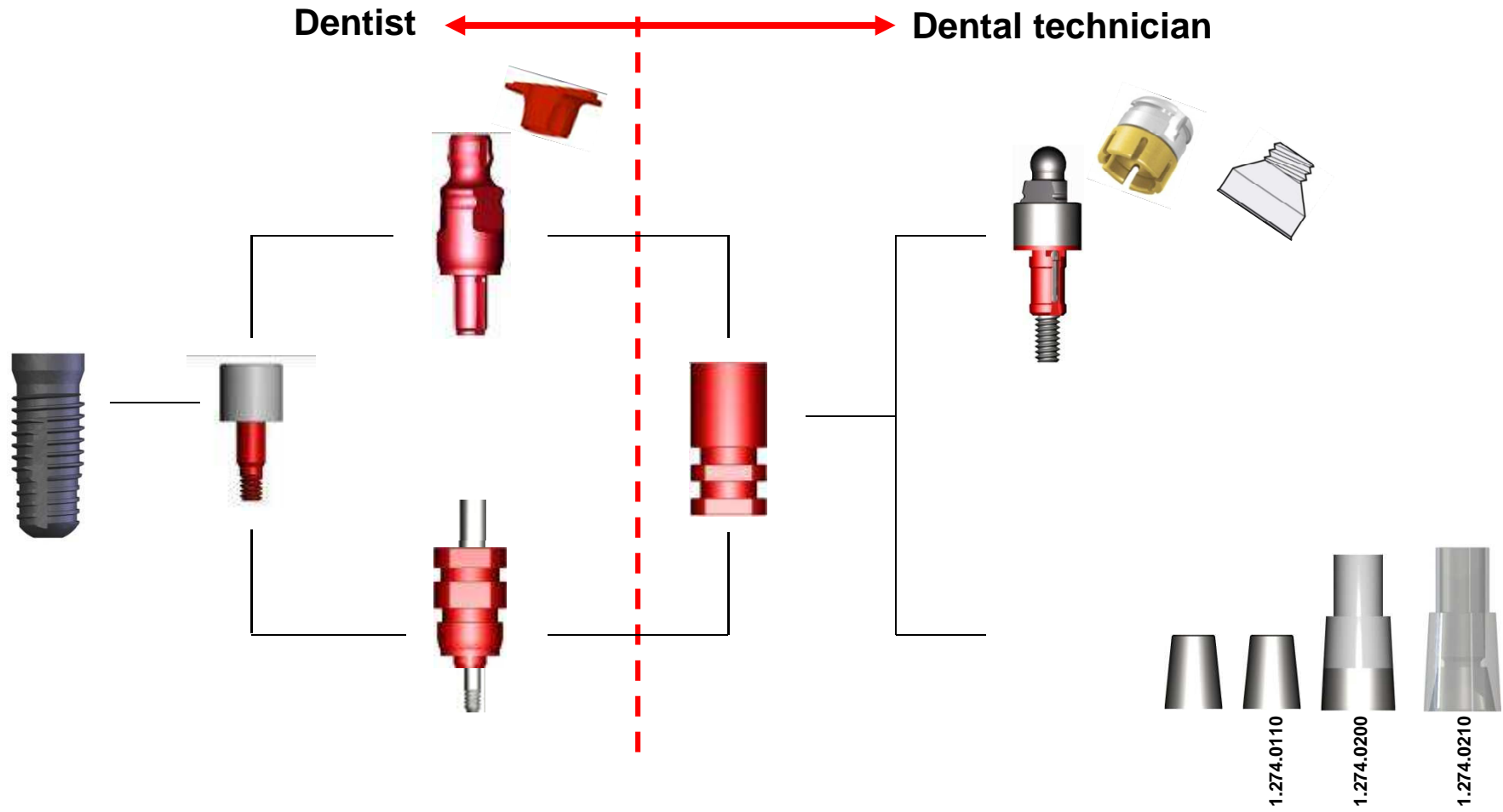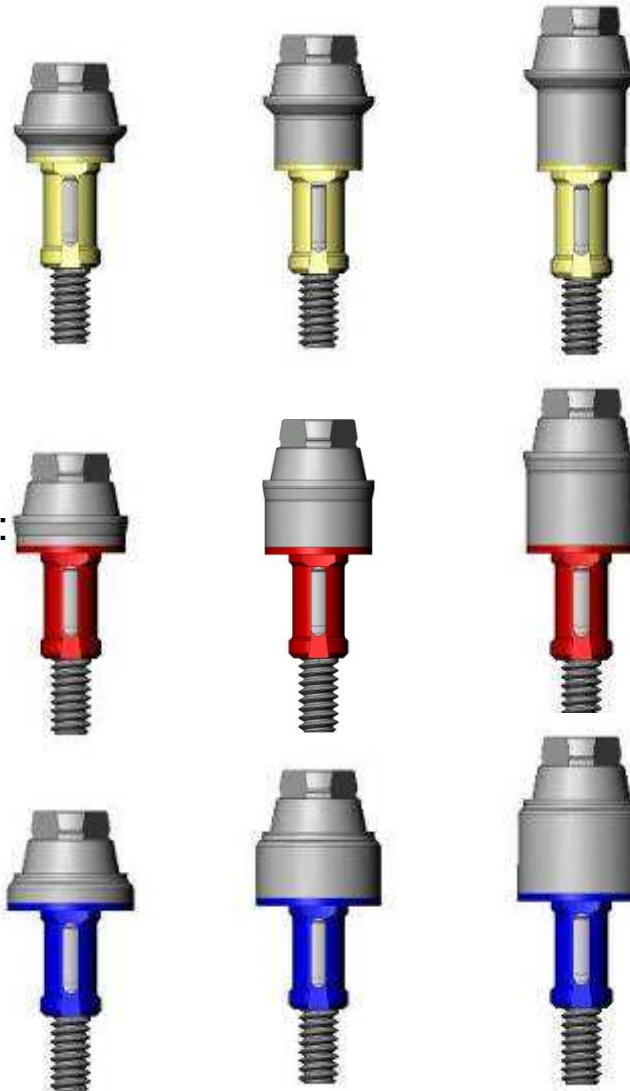
5,0

Gingiva height:  
1,5 mm  
3,0 mm  
4,0 mm


## Bar abutment direct after the implant healing time, without an healing cap

During the gingiva healing time, use the sealing cap 1.274.000

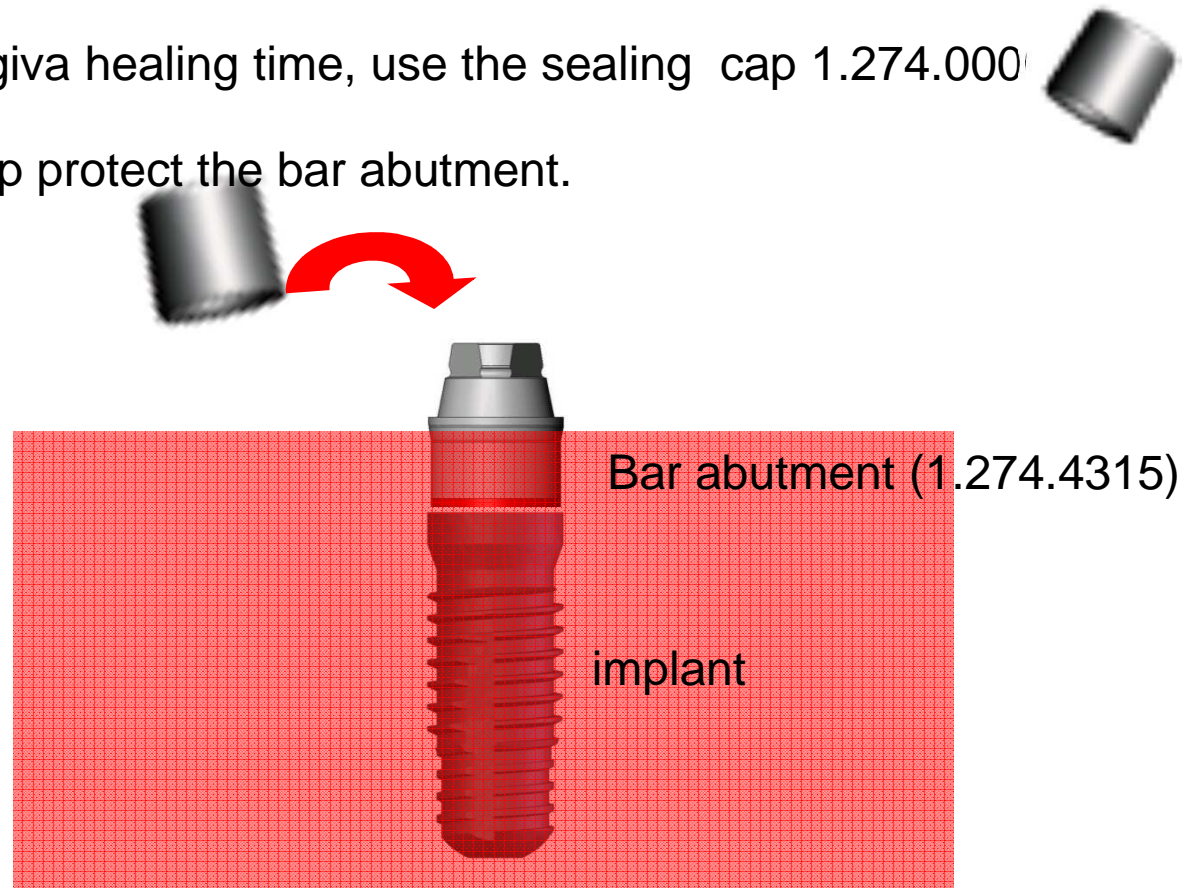The sealing cap protect the bar abutment.

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
## **Bar abutment direct after the implant healing time, without an healing cap**

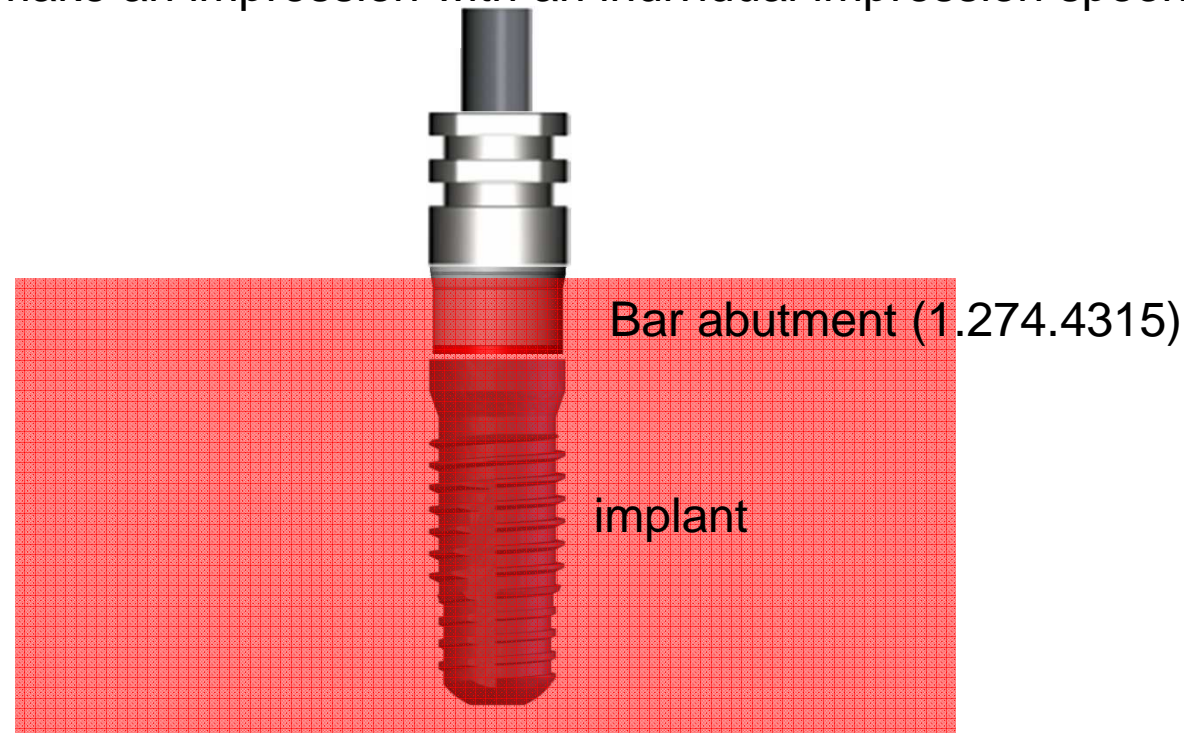after the gingiva healing time, use the impression coping for bar abutment 1.203.0000

Then make an impression with an individual impression spoon

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
## Bar abutment direct, if use the impression coping for impression

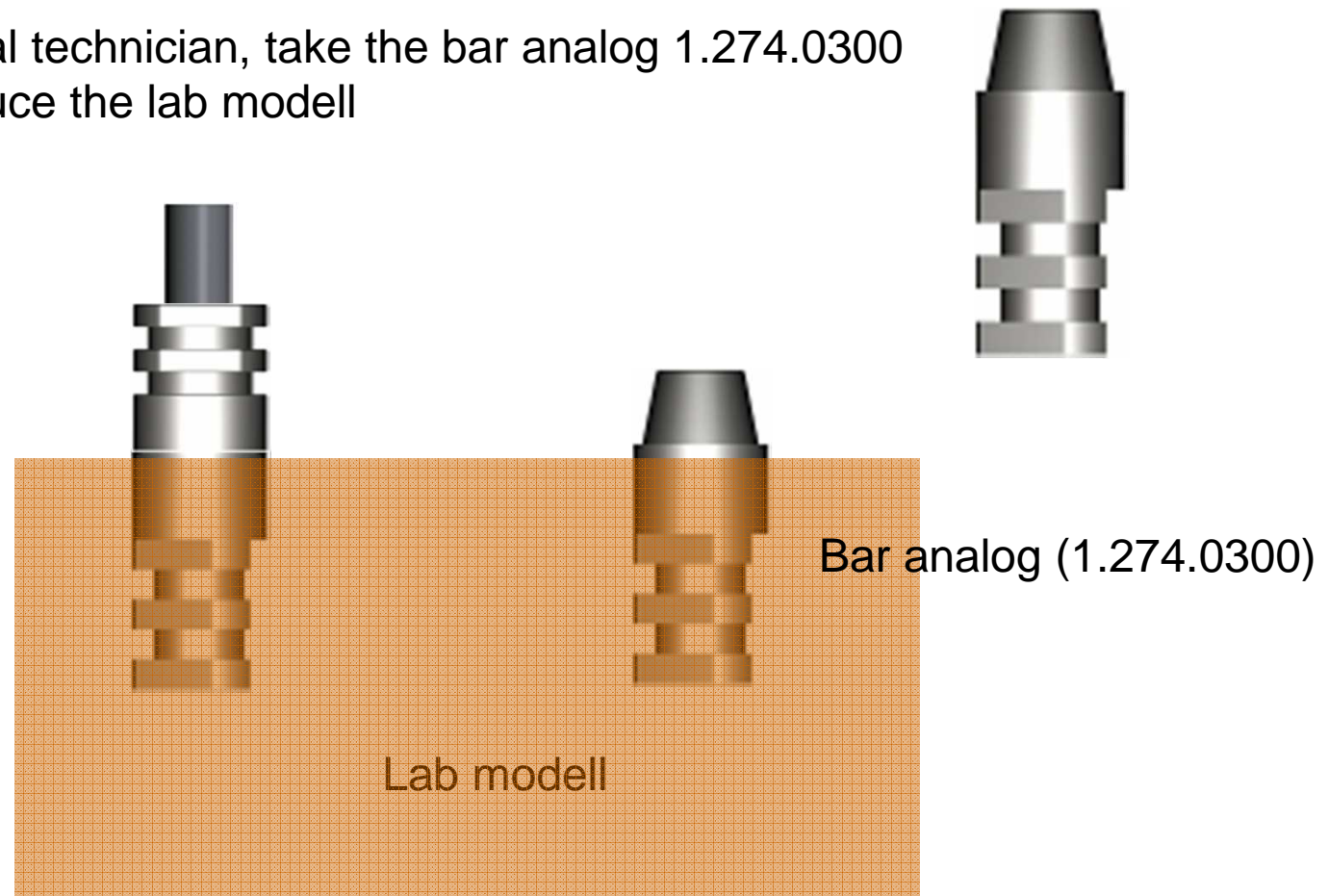The dental technician, take the bar analog 1.274.0300 and produce the lab modell

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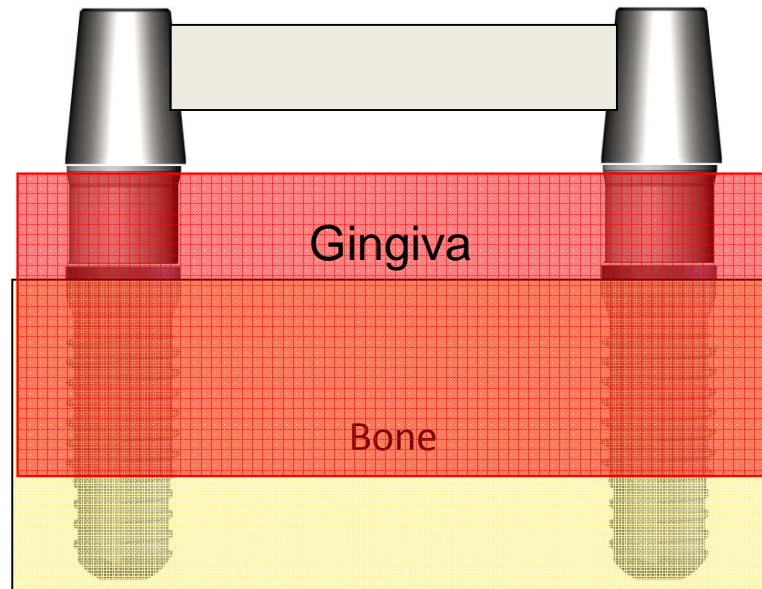
Minimum 2 Bar abutments /  
View: frontal

Bar: length is variable

Options: prefabricated or individuall

Bar base for diverse solution

Height for the Bar abutment =  
gingiva thickness

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
## The bar base can balance the divergence!

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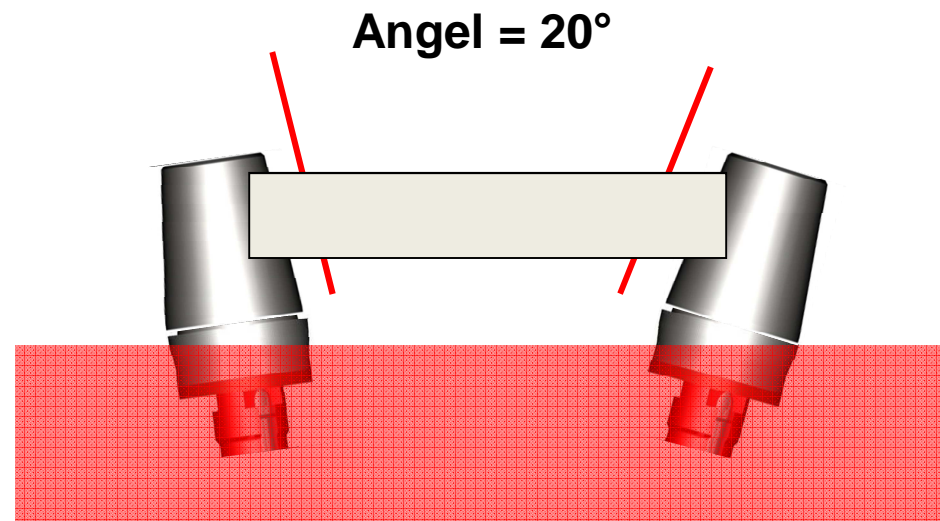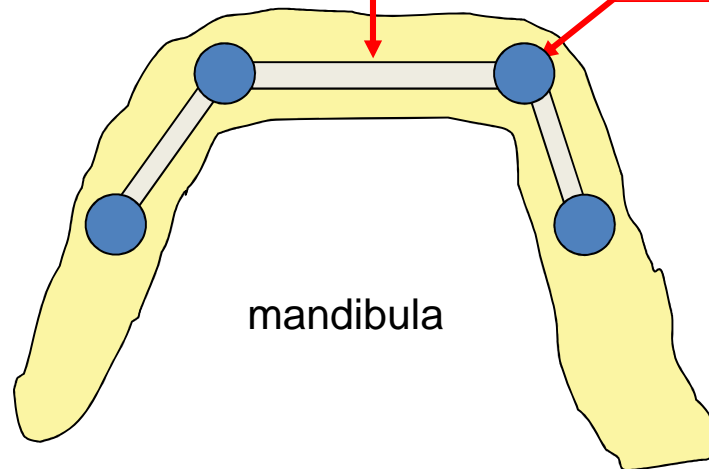
Minimum 2 Bar abutments /  
better for statics are 4 implants

View: occlusal

Bar: length is variable

Bar's prefabricated or  
individual

Implant, Bar abutment  
and bar base

Overdenture is fixed on the  
bar's  
The patient can the  
overdenture take out for  
cleaning

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
## Prefabricated bars in 2 shapes = fixed on the bar base

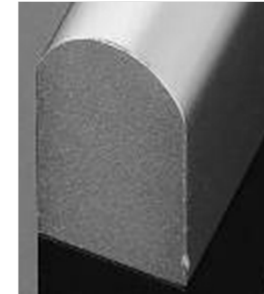Oval =  
The plaid  
can  
rotate

Steggelenk


Parallel

Steggesschiebe

## Plaid for bar's in different materials = fixed in the overdenture

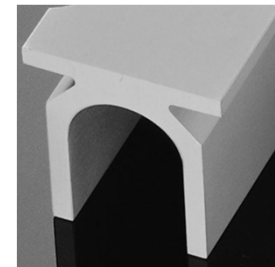plaid  
=  
Au- Pt

plaid  
=  
Titanium alloy